


BARIATRIC LEAK TEST MANOMETER

Catalog No. LTM-BA

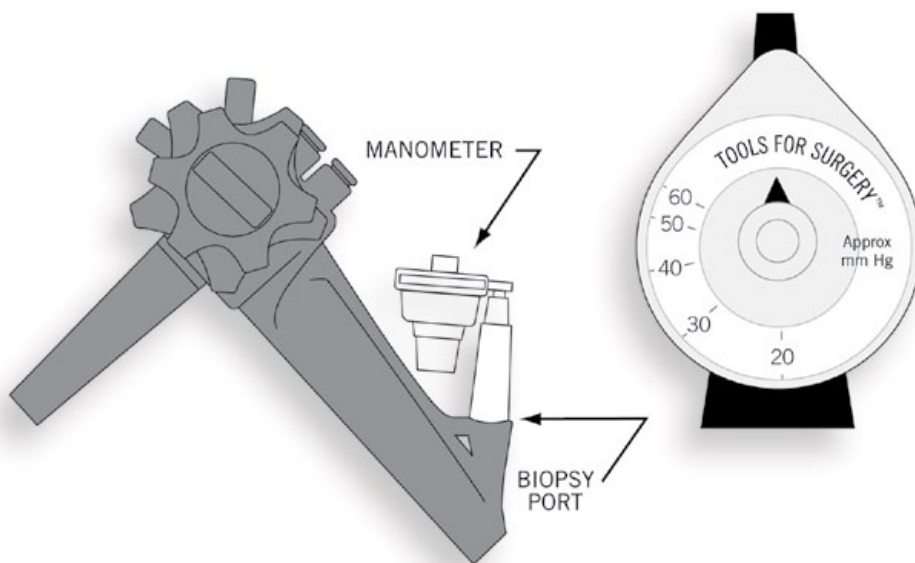

INDICATIONS:

For intraoperative leak testing of gastrojejunal anastomoses and gastric closures.

INSTRUCTIONS FOR USE:

TO TEST A GASTROJEJUNAL ANASTOMOSIS OR GASTRIC CLOSURE:

1. Immerse the anastomosis or closure in a pool of saline.
2. Make sure that the biopsy channel of the gastroscopy is patent.
3. Attach the BARIATRIC LEAK TEST MANOMETER to the biopsy port of the gastroscopy.
4. Perform endoscopy and insert the tip of the gastroscopy near the anastomosis or closure.
5. Inflate the stomach to the pressure desired for leak testing. The MANOMETER measures intraluminal pressure in mm Hg.
6. Observe the pool of saline for bubbles, which indicate an air leak.
9. Repair the anastomosis or closure if necessary and repeat the leak test.



Rx only

Clean - not sterile

Latex-free

TfS Tools
for
Surgery

We make it simple, by design.

Tools for Surgery, LLC
1339 Stony Brook Road ■ Stony Brook, New York 11790
PH (631) 444-4448 ■ FX (631) 444-5330

www.toolsforsurgery.com